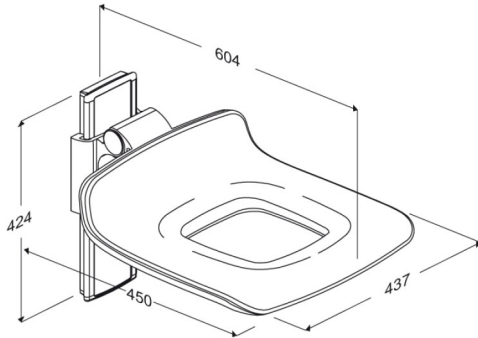




Product Item number	PLUS shower seat 450 with aperture, manually height adjustable R7411112000
Colour	VVS:777535200 NRF:6311158 EAN:5708590311333 HMI: 71804 NBS:N13:448 FDA:INN/D135865 White
Product description	 <p>PLUS shower seat 450 with aperture. Manually height adjustable by 220 mm. The seat is foldable. Anthracite cover plate for vertical wall track. Materials: Comfortable polyurethane foam over a plywood core, fitted to an aluminium profile. Excl. mounting kit.</p> <p>PLUS shower seat 450 with aperture. Manually height adjustable by 220 mm. The seat is foldable. Anthracite cover plate for vertical wall track. Materials: Comfortable polyurethane foam over a plywood core, fitted to an aluminium profile. Excl. mounting kit.</p>
Adjustability Adjustment height Product capacity	Height adjustable 220 mm Maximum user mass: 200 kg. When the seat is loaded with 200 kg, each wall fastener is loaded with 47 kg. <i>The seat has been tested statically with safety factor 1.5, which is a requirement according to the applied assistive products standards.</i>
Materials Ergonomy Features	The shower seat is made of polyurethane (PU) foam with a plywood core and this is covered with an in-mould coated surface that is protected using a clear PU lacquer. Internal components and fasteners are primarily made of stainless steel. The hinge barrel is capped with ABS. Ergonomically designed seat - contact surface are 450 x 437 mm. Soft and slip-resistant material that has a warm feel. The seat can fold up when not in use or to maximise floor space.



PRESSALIT®

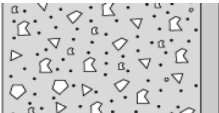
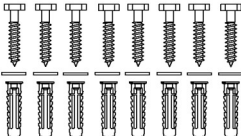

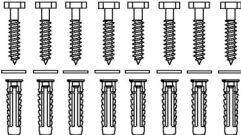
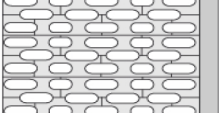
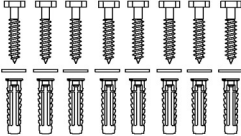
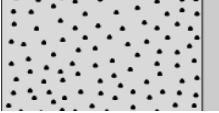
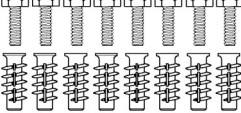
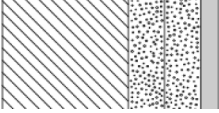
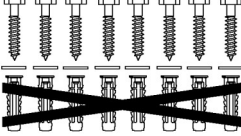
Product	PLUS shower seat 450 with aperture, manually height adjustable
Item number	R7411112000
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.
Cleaning	<p>PLUS shower seat.</p> <p>The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS: Erisan OXY+ Tvättsprit 70% CaviCide surface disinfectant decontaminant cleaner Rodalon Desinfektionsspray Terralin protect (SCHÜLKE+) Dismonzon Plus (BODE Chemie GmbH)</p> <p>NON-DISINFECTANTS: Wetrok Caledor (Wetrok) Petrosol Citro (Büfa)</p> <p>Important: Polyurethane surfaces of PLUS shower seats and PLUS backrests contains an antibacterial surface, which will be compromised if cleaned with anything other than soap and water.</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p>DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.</p>



PRESSALIT®

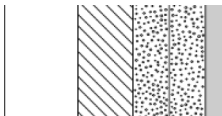
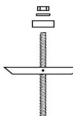

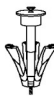
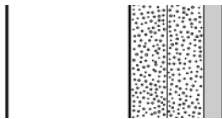
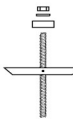

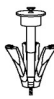
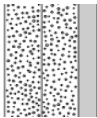
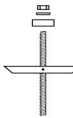


Product	PLUS shower seat 450 with aperture, manually height adjustable
Item number	R7411112000

Installation in different wall types

	<p>Concrete >B25</p> <p>V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs SX10. To be ordered separately.</p> 
	<p>Solid Brick >= Mz12</p> <p>V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs SX10. To be ordered separately.</p> 
	<p>Solid Sand Lime Brick >= Hlz12</p> <p>V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs SX10. To be ordered separately.</p> 
	<p>Aerated Concrete P4</p> <p>V8660 - Mounting kit, 8 countersunk bolts M8 x 35, plugs FTP M 8. To be ordered separately.</p> 
	<p>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</p> <p>V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs SX10. NB! Plugs are not needed. To be ordered separately.</p> 



PRESSALIT®

Product Item number	PLUS shower seat 450 with aperture, manually height adjustable R7411112000		
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.	  	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm) Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.	  	
	Drywall (2 x 12,5 mm) water-resistant Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.	  	

pressalit.com