

# PRESSALIT.

Product	PLUS shower seat 450, manually height adjustable		
Item number	R7420112000		
	VVS:777535400		
	NRF:6311163		
	EAN:5708590311517		
	HMI: 71801		
	NBS:N13:448		
	FDA: INN/D135865		
Colour	White		
Product description	PLUS shower seat 450, with backrest. Manually height adjustable by 360 mm. The seat is foldable. Anthracite cover plate for vertical wall track.  Materials: Comfortable polyurethane foam over a plywood core, fitted to an aluminic profile.  Excl. mounting kit.  PLUS shower seat 450, with backrest. Manually height adjustable by 360 mm. The seat is foldable. Anthracite cover plate for vertical wall track.  Materials: Comfortable polyurethane foam over a plywood core, fitted to an aluminic profile.  Excl. mounting kit.		
Adjustability	Height adjustable		
Adjustment height	360 mm		
Product capacity	Maximum user mass: 200 kg. When the seat is loaded with 200 kg, each wall fastener is loaded with 24 kg.		
	The seat has been tested statically with safety factor 1.5, which is a requirement according to the applied assistive products standards.		
Materials	The shower seat is made of polyurethane (PU) foam with a plywood core and this is covered with an in-mould coated surface that is protected using a clear PU lacquer. The backrest is made of polyurethane (PU) foam protected using a clear PU lacquer. Internal components and fasteners are primarily made of stainless steel. The hinge barrel is capped with ABS.		
Ergonomy	Ergonomically designed seat - contact surface are 450 x 437 mm. Soft and slip-resistant material that has a warm feel.		



## PRESSALIT.

Product	PLUS shower seat 450, manually height adjustable			
Item number	R7420112000			
Features	The seat can fold up when not in use or to maximise floor space. The seat and backrest can be adjusted simultaneously thanks to a safe and easy click system.			
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.			
Cleaning	PLUS shower seat. The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.			
	The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.			
	DISINFECTANTS: Erisan OXY+ Tvättsprit 70% CaviCide surface disinfectant decontaminant cleaner Rodalon Desinfektionsspray Terralin protect (SCHÜLKE+) Dismonzon Plus (BODE Chemie GmbH)			
	NON-DISINFECTANTS: Wetrok Caledor (Wetrok) Petrosol Citro (Büfa)			
	Important: Polyurethane surfaces of PLUS shower seats and PLUS backrests contains an antibacterial surface, which will be compromised if cleaned with anything other than soap and water.			
	The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.			
	DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that chalky deposits be removed from wall tracks and other			

aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts

must be wiped over with a damp cloth wrung out in clear water.



## PRESSALIT.

Product	

Item number

PLUS shower seat 450, manually height adjustable

R7420112000


### Installation in different wall types


la. V (( u . a)
2. 0. Z. R.
·
D C C C C
O . V . V . C

#### Concrete >B25

V8612 - Mounting kit, 10 coach screws Ø8 x 70, plugs SX10.

To be ordered separately.

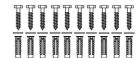

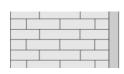


#### Solid Brick >= Mz12

V8612 - Mounting kit, 10 coach screws  $\emptyset$ 8 x 70, plugs SX10.

To be ordered separately.




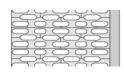

#### Solid Sand Lime Brick >= Hlz12

V8612 - Mounting kit, 10 coach screws  $\emptyset$ 8 x 70, plugs SX10.

To be ordered separately.

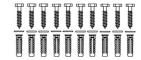





#### **Aerated Concrete P4**

V8662 - Mounting kit, 10 countersunk bolts M8 x 35, plugs FTP M 8.

To be ordered separately.

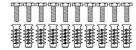


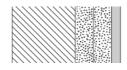

## Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood

V8612 - Mounting kit, 10 coach screws Ø8 x 70, plugs \$x10

NB! Plugs are not needed. To be ordered separately.

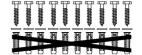



# PRESSALIT®

Product Item number	PLUS shower seat 450, manually height adjustable R7420112000			
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood  Wall fasteners (10 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.			
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm)  Wall fasteners (10 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.	01 0	Ç: UP	
	Drywall (2 x 12,5 mm) water-resistant  Wall fasteners (10 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.	81	<b>♀</b>	

pressalit.com