


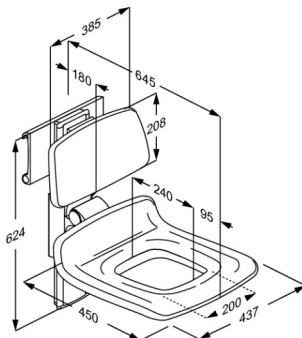

PRESSALIT®

Product Item number	PLUS shower seat 450 with aperture, manually height and sideways adjustable R7441182000
Colour Product description	White  PLUS shower seat 450 with aperture, with backrest. Manually height adjustable by 360 mm and sideways adjustable. The seat is foldable. Pale grey cover plate for vertical wall track. Materials: Comfortable polyurethane foam over a plywood core, fitted to an aluminium profile. For fitting to horizontal PLUS wall track (not included). PLUS shower seat 450 with aperture, with backrest. Manually height adjustable by 360 mm and sideways adjustable. The seat is foldable. Pale grey cover plate for vertical wall track. Materials: Comfortable polyurethane foam over a plywood core, fitted to an aluminium profile. For fitting to horizontal PLUS wall track (not included).
Adjustability Adjustment height Product capacity	Height and sideways adjustable 360 mm Maximum user mass: 200 kg. When the seat is loaded with 200 kg, each wall fastener is loaded with 23 kg. <i>The seat has been tested statically with safety factor 1.5, which is a requirement according to the applied assistive products standards.</i>
Materials Ergonomy	The shower seat is made of polyurethane (PU) foam with a plywood core and this is covered with an in-mould coated surface that is protected using a clear PU lacquer. The backrest is made of polyurethane (PU) foam protected using a clear PU lacquer. Internal components and fasteners are primarily made of stainless steel. The hinge barrel is capped with ABS. Ergonomically designed seat - contact surface are 450 x 437 mm. Soft and slip-resistant material that has a warm feel.



PRESSALIT®

Product	PLUS shower seat 450 with aperture, manually height and sideways adjustable
Item number	R7441182000
Features	The seat can fold up when not in use or to maximise floor space. The seat and backrest can be adjusted simultaneously thanks to a safe and easy click system.
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.
Cleaning	<p>PLUS shower seat.</p> <p>The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS: Erisan OXY+ Tvättspit 70% CaviCide surface disinfectant decontaminant cleaner Rodalon Desinfektionsspray Terralin protect (SCHÜLKE+) Dismonzon Plus (BODE Chemie GmbH)</p> <p>NON-DISINFECTANTS: Wetrok Caledor (Wetrok) Petrosol Citro (Büfa)</p> <p>Important: Polyurethane surfaces of PLUS shower seats and PLUS backrests contains an antibacterial surface, which will be compromised if cleaned with anything other than soap and water.</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p>DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.</p>



PRESSALIT®

Product	PLUS shower seat 450 with aperture, manually height and sideways adjustable
Item number	R7441182000

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com

PRESSALIT®