

## PRESSALIT.

<b>_</b>	DILLO has also at fine of has what
Product	PLUS backrest, fixed height
Item number	R6510000
	VVS:777538200
	NRF:6190615
	EAN:5708590314303
	HMI: 71810
	NBS:N13:426
	FDA: INN/D135865
Colour	White
	Materials: Aluminium, polyurethane foam with a plywood core. Incl. mounting kit for brick wall. PLUS backrest 385 mm, soft feel. For toilet without cistern, fixed mounting. Materials: Aluminium, polyurethane foam with a plywood core. Incl. mounting kit for brick wall.
Adjustability	Fixed mounting
Product capacity	100 kg.
Materials	The backrest is made of polyurethane (PU) foam which is covered with an in-mould coated surface that is protected using a clear PU lacquer.
Ergonomy	Soft and slip-resistant material that has a warm feel.
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.



## PRESSALIT.

Product	PLUS backrest, fixed height
Item number	R6510000
Cleaning	<ul> <li>PLUS backrest.</li> <li>The product is to be cleaned with hot water or a mild soapy cleaning agent.</li> <li>Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</li> <li>Important: Polyurethane surfaces of PLUS shower seats and PLUS backrests contains an antibacterial surface, which will be compromised if cleaned with anything other than soap and water.</li> <li>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</li> <li>DECALCIFICATION OF ALUMINUM PARTS:</li> <li>Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.</li> </ul>

## pressalit.com