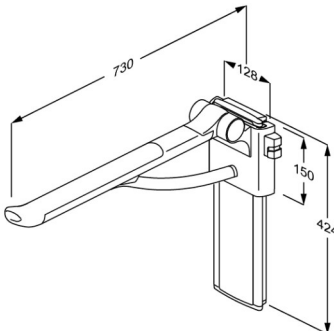




# PRESSALIT®

|   |   |
|---|---|
| <b>Product</b><br><b>Item number</b>  | PLUS support arm, height adjustable<br>R3170112000  |
| <b>Colour</b>   | VVS:615855000<br>EAN:5708590353357<br><br>White   |
| <b>Product description</b>  |  <p>PLUS support arm, 730 mm, ergonomically shaped to provide a clear sense of where the support arm ends. Right hand operated. Height adjustable by 240 mm without using tools. Detachable and foldable. Anthracite cover plate for vertical wall track. Materials: The arms are constructed of 10-micrometre anodized aluminium, with a non-slip gripping surface made of soft SEBS incorporating a glass-reinforced nylon core<br/>Excl. mounting kit.</p> <p>PLUS support arm, 730 mm, ergonomically shaped to provide a clear sense of where the support arm ends. Right hand operated. Height adjustable by 240 mm without using tools. Detachable and foldable. Anthracite cover plate for vertical wall track. Materials: The arms are constructed of 10-micrometre anodized aluminium, with a non-slip gripping surface made of soft SEBS incorporating a glass-reinforced nylon core<br/>Excl. mounting kit.</p> |
| <b>Adjustability</b><br><br><b>Adjustment height</b><br><br><b>Product capacity</b> | Height adjustable<br><br>240 mm<br><br>Maximum load: 150 kg.<br>When the support arm is loaded with 150 kg, each wall fastener in the vertical wall track is loaded with 41 kg.<br><br><i>PLUS support arms have been tested statically with 700 kg (close to the wall), 425 kg (in the middle) and 225 kg (the outermost point).</i>   |



|                    |   |
|--------------------|---|
| <b>Product</b>     | PLUS support arm, height adjustable   |
| <b>Item number</b> | R3170112000   |
| <b>Materials</b>   | <p>The arms are primarily constructed in 10-micrometre anodized aluminium, with a non-slip gripping surface at the end made of soft SEBS and incorporating a glass-reinforced nylon core.</p> <p>The hinge arm is also made of soft SEBS.</p> <p>Internal components and fasteners are primarily made of stainless steel.</p> <p>The hinge barrel is capped with ABS polymer.</p> |
| <b>Ergonomy</b>    | <p>The oval shape gives an easy and comfortable surface to lean and push against.</p> <p>The end of the support arm is enlarged so that it is easier to pull forward from a sitting position and also provides a clear sense of where the support arm ends.</p> <p>Soft and slip-resistant material that has a warm feel.</p>   |
| <b>Features</b>    | <p>Folds against the wall.</p> <p>The underside is shaped for the mounting of various accessories.</p>  |
| <b>Warranty</b>    | <p>Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.</p>  |



|                    |  |
|--------------------|--|
| <b>Product</b>     | PLUS support arm, height adjustable  |
| <b>Item number</b> | R3170112000  |
| <b>Cleaning</b>    | <p>PLUS support arms.</p> <p>The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b></p> <p>Clinimax Difficil-S<br/>Erisan OXY+<br/>Sprit (husholdningssprit)<br/>Suma Bac D10<br/>Virkon<br/>Rodalon Desinfektionsspray<br/>Lysoformin Spezial<br/>CaviCide<br/>Sufanios Premium<br/>Tvättsprit 70%<br/>Oxyplus-acid 2% (Diversey)<br/>WipeClean UniversalCleaning<br/>Chlorine Disinfection Wipes</p> <p><b>NON-DISINFECTANTS:</b></p> <p>Suma Star free D1<br/>Taski 100<br/>Taski 200<br/>Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p><b>DECALCIFICATION OF ALUMINUM PARTS:</b></p> <p>Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.</p> |



# PRESSALIT<sup>®</sup>

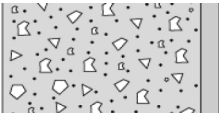
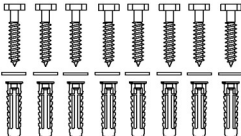

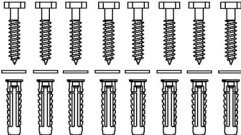
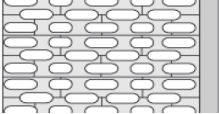
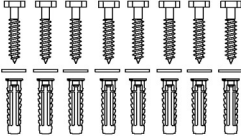
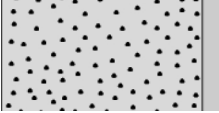
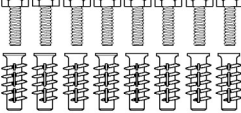
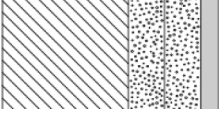
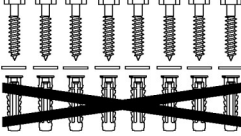
|                    |  |
|--------------------|--|
| <b>Product</b>     | PLUS support arm, height adjustable  |
| <b>Item number</b> | R3170112000  |
|                    | <p><b>Intended use</b><br/>The support arm is used as a support for sitting down or getting up from a toilet, or for sideways transfer from a wheelchair.</p> <p><b>Operating environment</b><br/>For indoor use in a wet environment, such as a bathroom or a washroom, in private homes, nursing homes and hospitals.<br/>For institutional use, the expected life time is 10 years.<br/><i>Life cycle test performed according to ISO 17966:2016.</i></p> <p>For maintenance, please refer to user guide, which is enclosed with the product, and which can also be found on the website under "Downloads".</p> |



PRESSALIT®

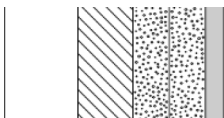
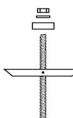

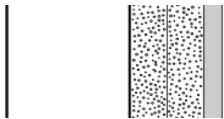
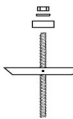

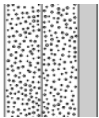
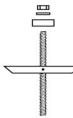

|                    |                                     |
|--------------------|-------------------------------------|
| <b>Product</b>     | PLUS support arm, height adjustable |
| <b>Item number</b> | R3170112000                         |

## Installation in different wall types

|   |   |   |
|---|---|---|
|    | <p><b>Concrete &gt;B25</b></p> <p>V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs SX10.<br/>To be ordered separately.</p>   |    |
|   | <p><b>Solid Brick &gt;= Mz12</b></p> <p>V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs SX10.<br/>To be ordered separately.</p>   |   |
|  | <p><b>Solid Sand Lime Brick &gt;= Hlz12</b></p> <p>V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs SX10.<br/>To be ordered separately.</p>  |  |
|  | <p><b>Aerated Concrete P4</b></p> <p>V8660 - Mounting kit, 8 countersunk bolts M8 x 35, plugs FTP M 8.<br/>To be ordered separately.</p>  |  |
|  | <p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</b></p> <p>V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs SX10.<br/>NB! Plugs are not needed.<br/>To be ordered separately.</p> |  |



# PRESSALIT®

| Product<br>Item number  | PLUS support arm, height adjustable<br>R3170112000   |   |   |
|---|--|---|---|
|    | <b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</b><br>Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.       |    |    |
|    | <b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm)</b><br>Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar. |    |    |
|  | <b>Drywall (2 x 12,5 mm) water-resistant</b><br>Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.                                       |  |  |

pressalit.com